



HELAGO-CZ, s.r.o.
Commercial Register maintained by the Regional Court in Hradec
Králové
Section C, File 17879
Kladská 1082
500 03 Hradec Králové 3
Company ID: 25 96 39 61, VAT: CZ 25963961
Phone: 495 220 229, 495 220 394
Fax: 495 220 154
GSM gate: 602 123 096
E-mail: info@helago-cz.cz
Web: http://www.helago-cz.cz

Fit-on Manual Disimpaction Simulator

Order code: **4109.M169**



Cena bez DPH

1.588,00 Eur

Price with VAT

1.921,48 Eur

Parameters

Quantitative unit

ks

Weight: About 4kg

Case size: W55 x D26 x H27cm

Accessories: Belt, Artificial excrement (@JPY2,000), Lubricant jelly (@JPY2,000), Baby powder, Storage case

Usage:

- Be sure to read "Precautions" prior to use.
- Wear rubber gloves during practice to keep the anal area clean.
- Put on the simulator with the help of an assistant.

Wearing the simulator:

- Fix the belt firmly to the rear of the simulator. Align the center of the belt with the center of the simulator.
- The person about to wear the simulator should lie on the practice bed with the knees raised.
- The assistant should take the simulator and apply it to the proper location of the person lying on the bed.
- The person with the simulator applied should ensure that it is in the proper location and secure it by fastening the belt.
- Begin the practice in a desired posture.

Setting the artificial tool

- Remove the cap from the top of the simulator. To remove the cap easily, pinch the cap at the recesses on both sides and pull it out.
- Take an appropriate volume of the artificial stool, and push and insert it into the simulator (the rectal area)
- Replace the cap on the simulator. Push in the cap so that it is aligned with the surface of the simulator.

Standard Package:

- Simulator
- Belt
- Artificial excrement
- Lubricant