

Web: http://www.helago-cz.cz

HELAGO-CZ, s.r.o. Commercial Register maintained by the Regional Court in Hradec Králové Section C, File 17879 Kladská 1082 500 03 Hradec Králové 3 Company ID: 25 96 39 61, VAT: CZ 25963961 Phone: 495 220 229, 495 220 394 Fax: 495 220 154 GSM gate: 602 123 096 E-mail: info@helago-cz.cz

## WA28395 - Injection Belly Order code: 4107.WA28395U

,20 Eur
42 Eur
Punkce, anestezie, injekce

Quantitative unit

This soft stomach replica will be an invaluable tool for teaching patients how to properly insert and rotate the infusion set for insulin pumps, avoid avoiding the 5 cm area around the navel, as well as giving self-injections of many types including pen-style injections. The trainer has lifelike skin with what feels like real stomach tis tissue underneath. There is a soft plastic backing to keep needles from going through and holes to allow the mate material to breathe and dry, should you choose to inject liquids. Distilled water may be injected, but the more liquid that is injected into the simulator, the longer the time for evaporation. Actual insulin or other medications should not be used, as they may deteriorate the material. The injection belly is considered disposable; however, with proper care and treatment it will serve its purpose for a long period of time. Using the small smallest needles possible when simu est simulating injections and rotating the insertion points will help prolong the life of the trainer. Dimensions:  $25 \times 15 \times 3$  cm.

ks