

NISSIM Type 1: Articulator

Order code: **4112.401184**



Cena bez DPH

302,00 Eur

Price with VAT

365,42 Eur

Parameters

Quantitative unit

ks

- Functionable and affordable middle grade simulator
- For use with FE, D and DP type articulators
- For beginning and intermediate students
- Face-bow training is possible
- Easy-to-use magnet change system

Training Method:

- Excellent durability
- Face bow training is possible
- Easy attachment of jaw models with magnet

Attachable jaw models:

- Jaw models with magnet plates and FE/D articulator can be attached onto NISSIM Type 1 articulator as they are.

Features of NISSIM Type 1:

- Some level of occlusal and jaw movement functions is possible. Amount of movement depends on the type of articulator attached
- Easy attachment using magnets. Models can be attached with the articulator
- One-touch easy removal of models

How to adjust occlusion on NISSIM Type 1?

- Loosen the adjusting screw in the back and slide down the stopper plate. Then attach the upper jaw
- Slide up the stopper plate so that the magnet sticks to the magnet plate of the lower jaw model
- Tighten the adjusting screw in the back

Type 1 Basic Set:

- NISSIM Head
- NISSIM Type 1 Articulator
- Mask:
 - Mask comes with a plastic tube, connector plug and drain stopper
 - The bottom of the mask has an open hole for attaching the connector plug
 - The tube and connector plug allows water draining
 - The drain stopper stops water draining
 - NISSIM Mask
 - NISSIM Mask with Water Drain (optional fee)