

Web: http://www.helago-cz.cz

HELAGO-CZ, s.r.o. Commercial Register maintained by the Regional Court in Hradec Králové Section C, File 17879 Kladská 1082 500 03 Hradec Králové 3 Company ID: 25 96 39 61, VAT: CZ 25963961 Phone: 495 220 229, 495 220 394 Fax: 495 220 154 GSM gate: 602 123 096 E-mail: info@helago-cz.cz

NISSIM Type 1: Articulator Order code: 4112.401184



Cena bez DPH Price with VAT

Parameters

Quantitative unit

- Functionable and affordable middle grade simulator
- For use with FE, D and DP type articulators
- For beginning and intermediate students
- Face-bow training is possible
- Easy-to-use magnet change system

Training Method:

- Excellent durability
- Face bow training is possible
- Easy attachment of jaw models with magnet

Attachable jaw models:

• Jaw models with magnet plates and FE/D articulator can be attached onto NISSIM Type 1 articulator as they are.

Features of NISSIM Type 1:

- Some level of occlusal and jaw movement functions is possible. Amount of movement depends on the type of articulator attached
- Easy attachment using magnets. Models can be attached with the articulator
- One-touch easy removal of models

How to adjust occlusion on NISSIM Type 1?

302,00 Eur 365,42 Eur

ks

- Loosen the adjusting screw in the back and slide down the stopper plate. Then attach the upper jaw
- Slide up the stopper plate so that the magnet sticks to the magnet plate of the lower jaw model
- Tighten the adjusting screw in the back

Type 1 Basic Set:

- NISSIM Head
- NISSIM Type 1 Articulator
- Mask:
 - $\circ~$ Mask comes with a plastic tube, connector plug and drain stopper
 - $\circ\;$ The bottom of the mask has an open hole for attaching the connector plug
 - $\circ\;$ The tube and connector plug allows water draining
 - The drain stopper stops water draining
 - NISSIM Mask
 - $\circ~$ NISSIM Mask with Water Drain (optional fee)