


HELAGO-CZ, s.r.o.
Commercial Register maintained by the Regional Court in Hradec Králové
Section C, File 17879
Kladská 1082
500 03 Hradec Králové 3
Company ID: 25 96 39 61, VAT: CZ 25963961
Phone: 495 220 229, 495 220 394

Fax: 495 220 154 GSM gate: 602 123 096 E-mail: info@helago-cz.cz Web: http://www.helago-cz.cz

Acupuncture Training Pad (With indicator light and buzzer)

Order code: 4109.AM-2

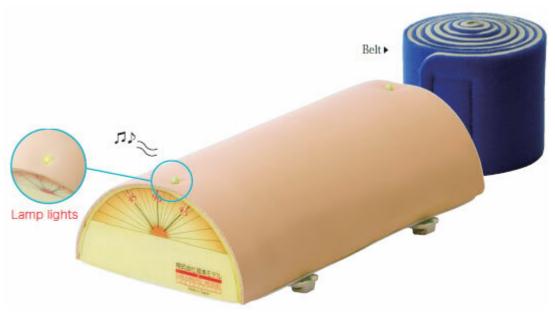

Cena bez DPH 1.006,00 Eur Price with VAT 1.217,26 Eur

Parameters

Quantitative unit ks

Two target points of different depth facilitate a practice by level of difficulty.

1. Practice by level of difficulty

- Two target points of different depth allows to practice by difficulty levels.
 The training pad has indicator light and buzzer sound to inform whether the point of insertion is correct or incorrect, it gives a sense of three dimensional imaging.
- Insertion to the shallow target point Attach the rubber band to the hook on the side of the pad, this confine the area, and makes it easier to locate the target point.
- Insertion to the deep target point.
- Indicator light and buzzer sound The green light turns on and a buzzer sounds when the needle is inserted to the target point. (The light does not turn on when inserted to the wrong point) The shape of sensor which detects the needle insertion is hemispherical, and it judges accurately whether insertion is perpendicular or oblique.

2. Practice by attaching the pad

• Hook and loop fasteners can be attached to any part of the body. Pair practice facilitates a realistic needle insertion practice in actual posture. More realistic practice is possible to the moving subject.

• Weight: about 0,3 kg

• **Case size:** 22 × 12 × 8 cm

• Accessories: Belt, Elastic band, Storage case